



MIP

The Softer Side of Healthcare

Swift[®] In-Bed Repositioning System Brochure



Affordable, Comfortable, Discreet

As a global supplier of healthcare textile solutions, MIP provides comfort and efficiency for the healthcare community and its people, based on the belief that a real, vital impact exists within the seamless integration of science and caring around the world.

INTRODUCING SWIFT®



Manual Handling Experts, Occupational Therapists, Nurses & Caregivers alike are clear, they want a repositioning solution that is affordable and easy to use. Introducing Swift® from MIP, the washable, in-bed system that is also discreet and comfortable for the care receiver.

Swift® can be used in community and residential care, respite units, hospice care and some acute settings such as frail elderly, complex needs, palliative care and intensive care.



Winner of the National Back Exchange Industry Innovation Award
For Promoting Excellence in Manual Handling




How Does Swift® Work?

The Swift® In-Bed Repositioning System features 2 elements; the UltraSlide® Bottom Sheet and the Swift® Slider, which work together to support repositioning and turning manoeuvres



UltraSlide® Bottom Sheet

A fitted bottom sheet modified with a low-friction taffeta panel down the centre.

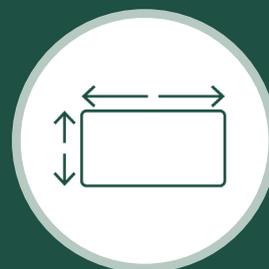

Swift® Slider

A friction reducing device that remains in-situ to assist with care receiver repositioning and turning in bed.

WHY SWIFT®?



The Swift® In-Bed Repositioning System should be used when a person has little or no mobility and requires extensive or total assistance for repositioning in bed.



Our durable and breathable WhooTex® fabric, supports longevity in demanding environments, as well as providing comfort to the care receiver.



Swift® remains on the bed at all times and benefits from a discreet design, to look like standard bedding when not in use (Swift® Slider flaps tucked in). The in-bed design also helps promote compliance.



Swift® reduces the force required to turn or reposition a bed bound patient by up to 60% and therefore reducing risk of injury to staff*.



*Please see Swift® User Guide

UltraSlide® Bottom Sheet

- Replaces your standard bottom sheet, available in standard and bariatric sizing.
- The stretch SleepKnit® construction means the UltraSlide® Bottom Sheet is flexible enough to fit most healthcare mattresses including profiling beds & alternating air pressure reducing mattresses.
- Carbon fibres integrated into the centre panel reduces static build up.
- Can also be used in isolation of the Swift® Sliders to encourage more independent movement.



Swift® Sliders

- Available in Standard Length (shoulder to hip coverage), Long Length (head-to-toe coverage) and Slider with Handles (head-to-toe coverage with handles to attach to a hoist if needed) all offering multi-directional movement.
- All 3 sliders are suitable for both standard and bariatric application.
- The low friction backing allows staff to reposition a person safely and easily whilst the breathable construction helps to avoid heat build up.
- The Swift® Slider's thin profile will not impair the performance of alternating air pressure reducing mattresses and can therefore remain on the bed when not in use (tucked in), making it readily available when needed.
- Our breakthrough WhooTex® fabric ensures a soft, comfortable feel against the skin.



Swift® Slider Long Length



Swift® Slider Standard Length



Swift® Slider with Handles

Swift® UltraSlide® Bottom Sheets

Product Code	Description	Colour	To Fit Mattress Size	Suitability
PTD-LS/S/UK	Swift® UltraSlide® Bottom Sheet Standard	White with blue centre panel	Width: 32" to 36" (81-91cm) Length: 78" to 84" (198-213cm) Depth: 4"-8" (10-20cm)	Designed for standard sized mattresses
PTD-LS/BA1/Y/UK	Swift® UltraSlide® Bottom Sheet Bariatric	White with blue centre panel (yellow binding for easy identification)	Width: 42" to 50" (107-127cm) Length: 78" to 84" (198-213cm) Depth: 5"-9" (10-20cm)	Designed for bariatric sized mattresses

Swift® Sliders

Product Code	Description	Colour	Size	Suitability
PTD-56/BA/Y/UK	Swift® Slider Standard Length	White with yellow low friction backing	L123 x W186cm	Designed for use with both standard and bariatric mattresses
PTD-56/BA/LG/UK	Swift® Slider Long Length	White with blue low friction backing	L200 x W205cm	Designed for use with both standard and bariatric mattresses
PTD-56/BA/H/UK	Swift® Slider with Handles	White with yellow low friction backing	L200 x W243cm	Designed for use with both standard and bariatric mattresses



LEARN MORE ABOUT SWIFT®



In-Bed Repositioning Learning Hub

- A learning platform by MIP to provide a range of digital education and training tools to support knowledge and learning in using in-bed repositioning systems such as Swift®.
- Tutorial videos, blogs, external resources and more.



Access this page by scanning the QR code above.



Swift® In-Bed Repositioning System User Guide

- An in-depth guide with detailed instructions on using the Swift® System.
- Closer look at some of the testing data and information that goes into the making of the Swift® System.



Access this guide by scanning the QR code above.

When undertaking any moving and handling tasks:

1. Make sure you have done your own risk assessment and that the technique being used is appropriate.
2. Or follow the handling plan from somebody who has done a risk assessment.
3. Check all equipment for any signs of wear and tear prior to use.

CONTACT US

mipuk.co.uk

UK 01525 218146

