

ardoo Caresafe 140 Hoist/Stand Aid Combo

User Manual



Manufactured By:
Polymorit Ltd
Unit 8 Oakhurst Business Park
Wilberforce Way,
Southwater, West Sussex
RH13 9RT, UK

Tel: +44 (0)330 6600 125
Email: sales@polymorit.com
www.polymorit.com

EU Authorised Representative:
Polymorit Ltd
51 Bracken Road,
Sandyford
Dublin D18 CV48
Ireland

Tel: +353 (0)1 906 9560

Inspect Your Merchandise

The product you purchased is guaranteed by Ardoo Caresafe Limited to be free from defects in material and workmanship under normal use and service. The warranty period is as stated in the Ardoo Caresafe 140 User Manual.

On receipt of the Ardoo Hoist/Stand Aid Combo, check that the product is free of any shipping damage. If the product is received damaged or missing components, contact Ardoo Caresafe immediately.

Safety Instructions

Warning!

Failure to heed this warning may result in damage to the product or serious injury to the operator and or user.

Read and understand the instructions in the Hoist/ Stand Aid Combo User Manual and Ardoo Caresafe 140 User Manual before using the product.

Note!

Important information concerning the product and/or its correct and proper usage follows.

Failure to use this device according to instructions may cause serious injury.

Never attempt to transfer a patient or resident whose weight exceeds the indicated maximum capacity for this device or any accessory.

Never leave a patient unattended during transfer.

Do not use without a complete understanding of safe and correct operation.

Do not use for the purpose of transportation over long distances.

Maintenance & Inspection

Refer to the Ardoo Caresafe 140 User Manual for maintenance associated with the hoist. The Ardoo Hoist/Stand Aid Combo kit requires minimal maintenance on an ongoing basis. However, to ensure safety & proper use, the following steps should be taken on a monthly basis.

Check all bolt/nut assemblies to make sure they are tight, and no wear & tear is evident.

Replace and tighten any worn assemblies prior to using the Stand Aid. Check the castors to make sure they are in working order and are secured firmly to the Stand Aid.

Replace any worn or damaged casters prior to using the stand Aid.




The Ardoo Caresafe 140 Stand Aid Kit is an assembly which incorporates a footplate and kneepad.

1. Footplate (See Picture 1)



Picture 1

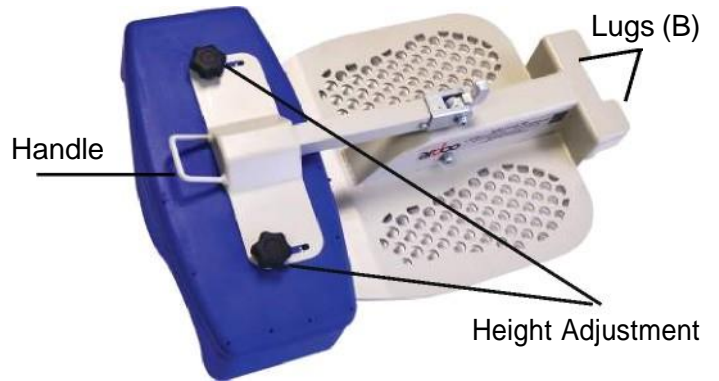

Picture 3

Converting the Ardoo Caresafe 140 Hoist into a Stand Aid:

Step 1:

Erect the Ardoo Caresafe 140 Hoist in accordance with the User Manual, paying special attention to the Safety Precautions.

Step 2:

The cross member of the Ardoo Caresafe 140 Hoist comes fitted with slots which are designed to take the Stand Aid Kit. See Picture 2



Picture 2

Cross Member with Slots (A)

Step 3:

Positioning the footplate onto the hoist to use as a Stand Aid.

The footplate is shipped in the folded position as per Picture 3.

Lift the footplate by the shaft and position the lugs (B) on the footplate into the slots (A) on the hoist cross member. Ensure the footplate fits firmly in place. See Picture 4.



Picture 4

Step 4:

Once the footplate is securely attached to the hoist grip the handle and slide the kneepad slightly forward to disengage it.



Lift the kneepad to a vertical position until you hear it click into place. See Picture 5.



Picture 5

When you have determined that Steps 1 — 4 have been followed the Stand Aid is ready for use by competent qualified persons.

To fold the kneepad down after use, place one foot on the front of the footplate, grip the handle and lift the kneepad slightly vertically to disengage it, lower the kneepad down to a horizontal position.

The footplate can now be lifted off the hoist for storage or to use the Combo as a hoist.

How to use the Stand Aid:

A suitable Stand Aid sling should now be fitted onto the user following manufacturer's instructions.

The hoist should be maneuvered as close as possible to the chair/wheelchair and back brakes applied.

Feet should be in tight fitting slippers or shoes, positioned flat on the foot plate, The lower legs should rest against the cushion. At this point the cushion can be adjusted for comfort.

The boom of the hoist should be lowered to facilitate the attachment of the stand aid slings to the hooks on the boom of the hoist.

The carer should now proceed with the lift by pressing the up button on the hand set. The carer can stand next to the person being aided and should check the knee pad for comfort and safety and the stand aid sling is securely fixed.



ardoo Caresafe 140 Hoist / Stand Aid Combo

