




Restwell Rise Recline Armchair

Installation and Set Up Procedure

Contents:

1x chair base, 1x back rest, 1x handset and key jack,
1x power supply, 1x mains lead

CHAIR INSTALLATION

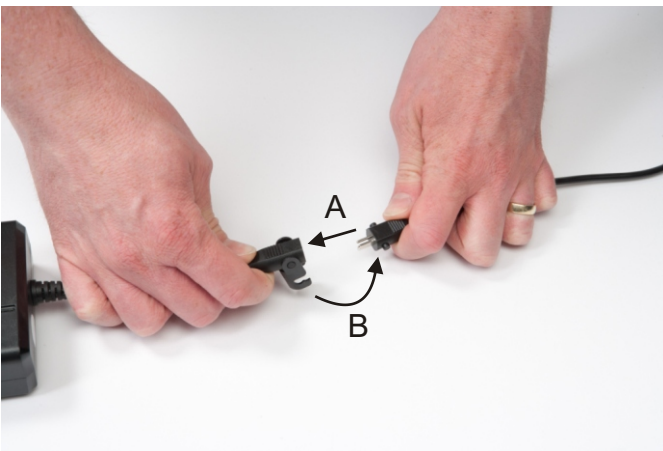


1. Remove the power supply from the pocket of the chair and locate the cable underneath the chair (from the chair actuator).

Plug the lead from the chair in to the socket mounted on the power supply cable (A). Then push the connector cover so it locks the connection (B).

2. Install the backup batteries if required (see separate section on installation).

Plug the figure-8 connector on the mains cable in to the power supply. Plug the 3-pin plug in to the mains socket and switch mains on.

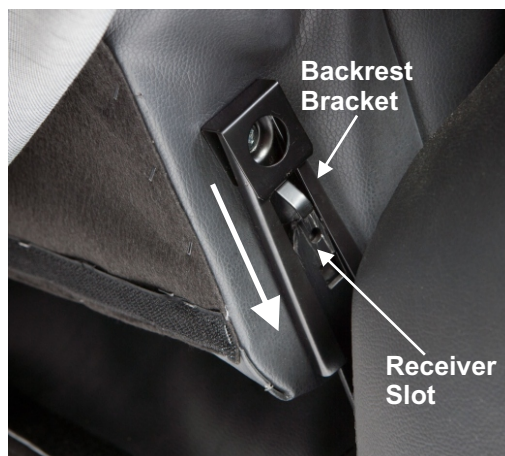

3. Position the backrest so its brackets sit on the receiver slot on both sides of the chair.

Push down on both sides of the chair until an audible click is heard.

Check both sides of the backrest are secured in place.

***** READ THE OWNER'S HANDBOOK FULLY BEFORE USE *****

BACK UP BATTERY INSTALLATION

Required: 2x 9V (PP3) batteries (not supplied)



1. Remove the battery compartment lid from the power supply by pushing the lid away from the power supply.



2. Pull out the two battery connectors from void in the power supply and push a 9V battery on to each of the connectors.



3. Place the batteries and wiring inside the void of the power supply.



4. Slide the battery compartment lid back on to the power supply.

BACKREST REMOVAL



1. Recline the chair to allow easier access to the backrest brackets.

2. Locate the brackets, which are covered by a leather flap.



3. Pull on the backrest whilst pulling out the locking tab (arrowed) with your finger or a flat end screwdriver. This will release the backrest and allow removal.

Repeat for the other side