ROHO® Standard Cushion Cover •

ROHO SELECT® Cushion Cover • ROHO Hybrid Elite® Cushion Cover
Instructions for Use (T25031)

 For additional information about using this cover, refer to the instructions provided with the compatible ROHO cushion.

Supplier: This manual must be given to the user of this product. Operator (Individual or Caregiver). Before using this product, read the instructions and save for future reference.

Incident Reporting: In the event of a serious incident or injury, contact Customer Support.

⚠️ Warning: Indicates that not following the specified instructions may lead to hazardous conditions resulting in serious injury.

⚠️ Warnings: Periodically, inspect the cover for damage and replace if necessary. • If you are unable to perform any steps, seek assistance. • The cushion surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the cushion will contact unprotected skin. • When possible, use hook and loop fasteners to further secure the cushion to the chair. Hook fasteners are available upon request. • If hook fastener strips were removed prior to washing, replace the strips.

Cover Placement:

1. Using the directional patch (A) for correct orientation, place the cushion inside the cover.

Make sure the air cells (B) face up and the non-skid bottom of the cover (Y) faces down.

The back (Z) of the cushion should face the zipper (D). The inflation valve (C) should face the front of the cover (X) and the carrying handle (E).

2. Close the zipper completely. The cushion should lie flat inside the cover with all air cells covered.

ROHO DRY FLOATATION® Cushions: The four corners of the cover are open, to allow access to the inflation valve(s) and other components (depending on the cushion model).

Intended Use: This cover is a textile enclosure intended to protect the cushion from solid debris.

*ROHO Standard Cushion Cover • ROHO SELECT Cushion Cover • ROHO Hybrid Elite Cushion Cover

In addition to the intended use statement, a clinician should be consulted regarding use of this cover within the individual's overall care regimen and including any need for large-print instructions or other assistive technology.

Compatibility: Compatible cushions are shown on the cover care label.

⚠️ Warnings: Periodically, inspect the cover for damage and replace if necessary. • If you are unable to perform any steps, seek assistance. • The cushion surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the cushion will contact unprotected skin. • When possible, use hook and loop fasteners to further secure the cushion to the chair. Hook fasteners are available upon request. • If hook fastener strips were removed prior to washing, replace the strips.

Cover Placement:


1. Using the directional patch (A) for correct orientation, place the cushion inside the cover.

Make sure the air cells (B) face up and the non-skid bottom of the cover (Y) faces down.

The back (Z) of the cushion should face the zipper (D). The inflation valve (C) should face the front of the cover (X) and the carrying handle (E).

2. Close the zipper completely. The cushion should lie flat inside the cover with all air cells covered.

ROHO DRY FLOATATION® Cushions: The four corners of the cover are open, to allow access to the inflation valve(s) and other components (depending on the cushion model).

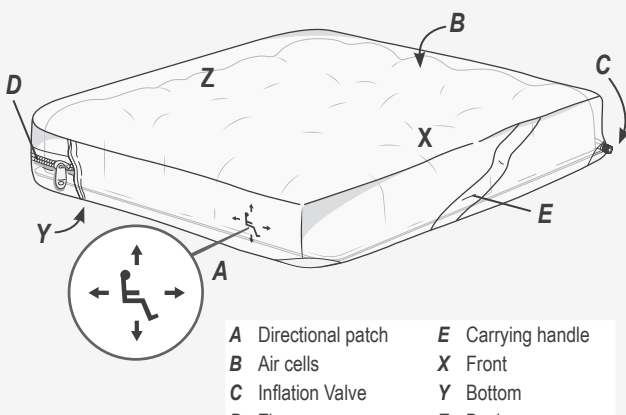Cover Removal: Unzip and remove the cover from the cushion.

Cover Care:

⚠️ Warnings: Clean the product regularly and when it becomes soiled. If the product becomes soiled, or between use by different individuals, first clean and then disinfect. Check the product for proper functioning. • Follow all manufacturer bleach container safety guidelines. • This product does not require sterilization; DO NOT autoclave.

Notes: Remove any hook fastener pieces from the bottom of the cover prior to cleaning and disinfecting. • Wash with like colors.

To Clean: Remove the cover. Machine wash warm with mild detergent, gentle cycle, or wipe clean with neutral detergent and water. Rinse thoroughly. Tumble dry low or air dry.

To Disinfect: Hand wash, using 1 part household liquid bleach per 9 parts water. Keep the product wet with the bleach solution for 10 minutes, then rinse thoroughly. Or machine wash in hot water (60°C). Tumble dry low or air dry.

Note about sanitization: Washing machines certified to NSF Protocol P172 are designed to heat water to 60°C. Recommendation: When washing by hand, use room-temperature water.

REF	Model number	WDH	Size (width x depth x height)	CL	Cleaning instructions	W	Machine wash warm, gentle.	H	Hand wash.
FITS	Fits	M	Date of manufacture	DI	Disinfecting instructions	W	Machine wash hot, normal, at the temperature shown.	T	Tumble dry low.
LOT	Batch code	✓	Meets flammability	M	Manufacturer	T19	Bleach (1 part bleach: 9 parts water).	MD	Medical Device
i	Consult instructions for use	EC REP	Authorized representative in the European Community	CH REP	Authorized representative in Switzerland				

Cover Labels: Two-way stretch top, spacer fabric sides; non-skid bottom; with a zipper located in the back of the cover. Refer to the cover care material for material content and for compliance to flammability requirements. Cover expected lifetime: 1 year

Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incinerate with other healthcare waste. Incineration must be performed by a qualified, licensed waste management facility.

Limited Warranty Term: 6 months; see also the Limited Warranty Supplement provided with the cushion, or contact Customer Support.

Products may be covered by one or more U.S. and foreign patents and trademarks, including DRY FLOATATION®, Hybrid Elite®, SELECT® and ROHO®.

ROHO, Inc. is a part of Permobil.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

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ROHO, Inc. face parte din Permobil.